



Germo®
S.P.A.

Capitale Sociale € 156.000,00 interamente versato - R.E.A. 497966 Cod. Fisc. e Part. IVA 00772350153 - Iscr. 125750 R.I. Canc. Trib. MI

Sede Legale Stabilimento e Uffici: 20032 CORMANO (Milano) Via Giotto, 19/21
Telefono +39-0266301938 r.a. - Telefax +39-0266301939
E-mail: info@germodis.com www.germo.it

GERMOCID LINE – TECHNICAL DATASHEET

GERMOCID BASIC

GERMOCID BASIC is a Medical Device CE 0546 class IIa in compliance with Directive 93/42/EEC.

GERMOCID BASIC is a ready-to-use disinfectant solution containing 60% of alcohol; GERMOCID BASIC is a wide range disinfectant and a great cleaner for surfaces and non-invasive medical devices.

The association of different ingredients of GERMOCID BASIC guarantees a rapid and complete disinfectant action, at the same time it deodorizes and cleans even in presence of organic materials.

GERMOCID BASIC is used for the treatment of all surfaces of non-invasive medical devices (facial masks, desks, armrests, headrests etc.).

GERMOCID BASIC is active in five minutes of contact against Gram-positive and Gram-negative bacteria, fungi (yeasts and molds) and viruses (Adenovirus, HIV, HBV, HCV); against mycobacteria (TBC included) in 60 minutes.



INSTRUCTIONS FOR USE

Shake well before use and spray GERMOCID BASIC directly on the surface to be treated. Let it act and finally dry it with a cloth or disposable paper. GERMOCID BASIC is active in five minutes of contact against Gram-positive and Gram-negative bacteria, against fungi (yeasts and molds) and viruses (HIV, HBV, HCV included); against mycobacteria (TBC included) in 60 minutes.

PACKAGING

Internal Code:	Primary packaging:	Secondary packaging:
R600	750 ml bottle	Box with 12 bottles
R602	1000 ml bottle	Box with 12 bottles
R605	5000 ml canister	Box with 4 canisters

All primary packaging is made according to the specifications provided by the Pharmacopoeia. The materials do not contain latex and are perfectly compatible with all components of the formulation.



Germo[®]
S.P.A.

Capitale Sociale € 156.000,00 interamente versato - R.E.A. 497966 Cod. Fisc. e Part. IVA 00772350153 - Iscr. 125750 R.I. Canc. Trib. MI

Sede Legale Stabilimento e Uffici: 20032 CORMANO (Milano) Via Giotto, 19/21
Telefono +39-0266301938 r.a. - Telefax +39-0266301939
E-mail: info@germodis.com www.germo.it

GERMOCID LINE – TECHNICAL DATASHEET

GERMOCID BASIC

COMPOSITION

100,0 g of product contain: Chlorhexidine Digluconate 0,1 g; Benzalkonium chloride 0,1 g; Ethanol 45,0 g; Isopropyl Alcohol 15,0 g; Coformulants, depurated water q.s. to 100,0 g.

PRODUCT FORM

Colourless liquid without added perfume.

ACTIVITY

Chlorhexidine Digluconate in combination with Quaternary Ammonium Compound and the high content of Alcohol guarantees a wide spectrum of action against Gram-positive and Gram-negative bacteria, mycobacteria (TBC included), fungi (yeasts and molds) and viruses (Adenovirus, HIV, HBV, HCV).

TEST

EN13727; EN13624; EN14348; EN13697; EN14561; EN14562; EN14476.

WARNINGS

Avoid using the product on material sensitive to alcohol.

VALIDITY

3 years. The indicated period of validity refers to the product stored in its container and properly used and stored.

HOW TO STORAGE

Store in a cool, dry place, away from heat sources.

QUALITY CONTROL

The components (raw materials, containers, labels, etc.) and the processing steps of each lot are regularly and carefully checked internally following the procedures of the Organization Quality Management System certified UNI EN ISO 9001 and UNI EN ISO 13485.

AUTHORIZATIONS

Medical Device CE 0546 class IIa in compliance with Directive 93/42 / EEC as amended by Directive 2007/47 / EEC.

CONFIDENTIAL INFORMATION FOR PROFESSIONAL USERS